



GoTo


Factsheet #04:

Attaching the GoTo Seat to the Floorsitter accessory

Attaching the GoTo Seat to the Floorsitter accessory



Before attaching the GoTo Seat to the Floorsitter accessory, follow steps in section 10(b).

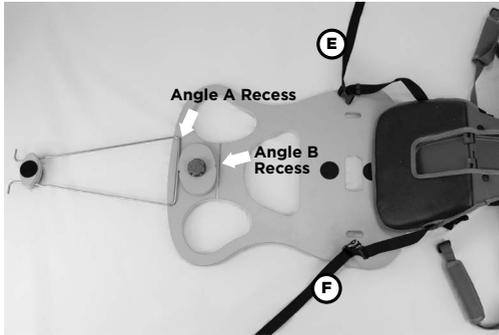

- 1 The Angle A Recess (20°) will provide more tilt than the Angle B (10°). Swing the metal prongs to the rear of the Floorsitter and align the GoTo at the front of the Floorsitter as pictured above.

- 2 Thread straps E and F through the two narrow holes at the front of the Floorsitter.



- 3 Turn the Floor sitter around to fasten these straps underneath.



- 4 Pull straps E and F to tighten the GoTo against the Floorsitter. Slide the tidy clips out of the way so that they are not on the base of the Floorsitter.

Attaching the GoTo Seat to the Floorsitter accessory



- 5 Ensure that all the straps are unfastened. The knob pictured is the support for attachment on to the GoTo. Lower the knob slightly by unscrewing it anticlockwise.




- 6 With this knob lowered/removed to allow movement of prongs, place the prongs of the Floorsitter metal support into their corresponding holes in the GoTo Seat base. Once threaded through the holes, place the prongs into their clips.



- 7 Re-adjust the knob and tighten it by turning it clockwise. This will lock the metal support frame securely in to the backrest.



- 8 Fasten straps A,B,C and D across the back of the prongs (and tidy as pictured above).

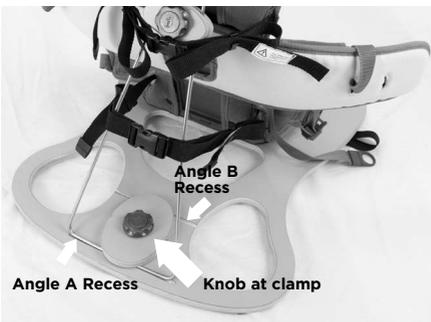

- 9 To adjust the angle of the Floorsitter, unscrew the knob at the Floorsitter clamp. Place the frame into the alternate recess and fasten the knob by screwing clockwise. Readjust straps A,B,C and D as needed.



Do not adjust the angle of the GoTo when the child is sitting in the GoTo.

- 10 To remove GoTo from the Floorsitter, carry out these steps in reverse order.