



SCOOT

Case Study – Ella



Introduction

Ella is an adorable, independent, and outgoing 5 year old girl who was diagnosed with Spinal Muscular Atrophy (SMA) Type II just after her first birthday. SMA is a genetically inherited neuromuscular condition, it can present as muscle weakness and low tone. Ella is able to roll and scoot at times, however she requires adult assistance to move on the floor or move from room to room in her home. She has used a gait trainer in physical therapy sessions but does not have one at home or in school.

As a result, she uses a powered wheelchair as her main way to explore her environment. In physical therapy Ella is working hard to improve her overall strength, head control, posture, range of motion and balance in order to maximize her independence at home, in school, and out in the community. Ella used the Scoot during many physical therapy sessions in the outpatient setting and had great success with the product!

Scoot Configuration

Ella trialed the Scoot at the Abilities Expo in Boston and was very excited to use it again at her initial trial with her physical therapist. Ella has very limited active movement of her lower extremities so she used the Scoot in a seated position using her arms to move around the therapy gym. She used the Scoot to manoeuvre around the therapy room during an Easter egg hunt which she thoroughly enjoyed. Although moving the Scoot was a challenge for Ella she was able to have fun with floor play, by exploring her environment all by herself! The Scoot increased Ella's access to toys placed on the floor. Alternative mobility devices like her power chair and gait trainer would not have allowed Ella to access toys on the floor.



Crawl Configuration

In the crawl configuration, Ella was in prone (tummy time) and was able to work on her head control, weight bearing through her knees, as well as assisted crawling. Ella worked hard on transitioning from a seated position into prone on the Scoot and back to a seated position. With assistance from myself (PT) to advance the Scoot forward Ella was able to reciprocally move her lower extremities in a crawling pattern.

This is a movement that Ella was not able to perform prior to using the Scoot. She was very excited and proud of her accomplishment - she even asked her mother to take a video of her so she could show her father later that evening. Overall Ella enjoyed using the Scoot during physical therapy sessions. It was a great tool to assist her in meeting her physical therapy goals as well as increasing her functional mobility/independence.



The Result

The Scoot gave many of my patients the chance for independent exploration of their environment that they are typically unable to do without equipment or caregiver assistance. In addition it allowed them to explore their environment on the floor, something that is very difficult or impossible for many children that I treat. The versatility of this piece of equipment and ease of changing it from the Scoot (seated position) to crawl (prone position) was very helpful in a busy outpatient setting. Additional support can be given or taken away from the child as needed.

Another feature of the Scoot that I enjoyed was that it can be used with a wide variety of children with varying abilities. I found that individuals who are unable or struggle to use a traditional scooter board were able to use the Scoot. Overall this product was a great addition to our clinic! Therapists, parents, and most importantly the children enjoyed incorporating the Scoot into our weekly sessions to help meet their therapy goals.

I would like to thank Firefly for giving me the opportunity to trial the Scoot in the outpatient clinic setting.

Danielle O'Shea PT, DPT
Outpatient Based Physical Therapist

www.fireflyfriends.com

Firefly, 19C Ballinderry Road,
Lisburn, BT28 2SA, Northern Ireland
+44 (0) 28 9267 8879

